

The SoClean 2

The SoClean 2 Unit will clean and sanitize CPAP equipment, including accessories such as the mask, hose, and the water reservoir. This is efficiently done without needing to disassemble any parts. Using activated oxygen, the SoClean 2 Unit will sanitize your CPAP machine daily with minimal effort on your part. The SoClean 2 Unit is compatible with ALL CPAP, APAP, and BiPAP units and systems. With the SoClean 2 Unit, you can know with certainty that your CPAP machine will be clean and will never make you sick.

The SoClean Sanitizing Process

The SoClean 2 Unit generates activated oxygen which is pumped into the humidifier reservoir through the hose and fitting that is connected to the CPAP system.

The activated oxygen sanitizes the water reservoir and water within. The oxygen then travels through the main hose, eliminating germs and impurities as it goes.

Activated oxygen passes through the CPAP mask and is released into the sealed chamber that holds it. In this manner the interior and exterior of your mask is exposed to the activated oxygen and thoroughly sanitized.

After passing through a filter, the activated oxygen is then released by the SoClean 2 Unit which converts it back into the oxygen we breathe.

All activated oxygen generated by the cleaning process is safely contained within the closed SoClean 2 Unit system. Neither the outside environment, nor the CPAP's electronics are ever exposed.